



MATERNITY SUPPORT BELT

The *Soft Form*™ line combines proven orthopaedic design with modern fabrics and latex-free elastics to provide the ultimate form and comfort. Each raw material component was specifically chosen to provide the best function, feel and appearance.

Soft Form™ products were all engineered in accordance with fundamental orthopaedic principles and designs.



"Fashioned for Form and Function"™



Latex Free

MODEL 31-570

Maternity Support Belt

Sizes: Petite, Universal (Med.), Large
Color: Beige

INDICATIONS:

Helps relieve the minor lumbar and abdominal pain and discomfort associated with pregnancy.

- Muscle Strain
- Lower Back Pain and Discomfort
- Abdominal Support

BENEFITS:

Comfortably supports the lumbar and abdominal regions to help provide relief of lower back pain and discomfort. Unique multi-layered soft laminate provides excellent support and comfort. Ventilated, latex-free elastic compression side panels provide compressive support. Lightweight flexible stays help keep the support in place and prevent rolling. Easy to wear under most clothing.

PRODUCT FEATURES:


- **Abdominal Lift Attachment** - Provides abdominal lift with no undesirable pressure. Expands to accommodate growth during various stages of pregnancy.
- **Ventilated Elastic Back and Side Panels** - Latex-free ventilated elastic provide comfortable and cool support compression.
- **Unique Laminate Material** - Combines three materials: durable outer in an attractive beige color, support foam base and soft cotton inner lining.
- **Easy Hook and Loop Closure** - Easy application.
- **Two Anti-Roll Stays** - Precisely placed stays help keep the support in place and prevent rolling.

WEARING INSTRUCTIONS:



1. **Wrap the Abdominal Lift Attachment around your waist.** Position the pad so it is centered and under the abdomen. Wrap the elastic strap around your body and attach to the pad. Both sides of the strap can be adjusted for a good fit.

Ensure the abdominal pad is properly positioned, but does not interfere while walking or sitting. The fit of this pad can expand as your baby grows to ensure proper fit. Simply fasten further away from the center of the pad on the Abdominal Lift Attachment.

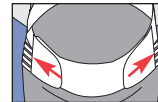

2. **Attach the belt to the pad.** With the belt centered on the lower back, pull both elastic sides until the desired tension and attach the closures to the abdominal pad. Ensure the "hook" ends of the belt have completely engaged to the pad to prevent skin irritation or snagging of clothing.



3. **To help keep the support in place and keep from sliding down,** stretch the top strap over your stomach and attach to the opposite side.



4. **For even greater lift and support,** with the belt on, lay on your back with your knees up. Unfasten the belt and retighten the elastic strap and reattach closures.

Your Soft Form™ Maternity Support Belt is now properly fitted and is easily and inconspicuously worn under most clothing.

Sizes:

Dress Size Before Pregnancy

Petite	2 - 6
Universal (Med.)	6 - 14
Large	16 - 20

Care Instructions:

Fasten all closures. Hand wash in cool water with mild soap. Remove excess water and air dry away from heat. Wash as needed.

Material Content:

40% Polyester, 25% Nylon, 15% Polyurethane, 10% Neoprene, 5% Lycra, 5% Cotton.

CAUTION:

**Consult your physician before wearing this product.
Inform your physician of any unusual or consistent
pain and discomfort.**



*Discover the full Soft Form™ line
of quality orthopaedic products
“Fashioned for Form and Function”™*



FLA ORTHOPEDICS, INC.

©2000 FLA ORTHOPEDICS, INC.

Part No. B5-031570